



www.alcosense.com

Now Group UK Ltd, Unit 3, Maidenhead Enterprise Centre,
Cordwallis Street, Maidenhead SL6 7BE. UK

Helpline: 0800 195 0088 (+44 1628 778885)

Instructions for Use

3 Line Tester Manual Version 2.1

Please Note: Before using this product you agree to have read and understood the instructions and are aware of the legal alcohol limit in your jurisdiction. The manufacturer distributors and sellers of this product exclude all liability for improper use or loss or damage caused by actions taken by you after use of this product. This product is non-returnable for hygiene reasons. This does not affect your statutory rights.

The AlcoSense Single disposable tester uses specially formulated crystals which change green in the presence of alcohol. Using the blow bag a measured volume of breath is passed through the tube, the length of the colour change from yellow to green indicates your alcohol level.

Wait at least 20 minutes after drinking and smoking before using the test. **Please note your blood alcohol level could continue to rise for up to 90 minutes after drinking alcohol. As a result it is best to use this test for the 'morning after'.**



- 1a)** Break off the two knurled plastic ends of the tube.
1b) Pour the white crystals into a bin.



- 2)** Blow up the bag in ONE breath until the bag is fully inflated.



- 3)** Insert the WHITE end of the tube firmly into the bag.



- 4a)** Gently and consistently squeeze the bag until all of the air has passed through the tube.
4b) Wait 2 minutes and review the results.

Instructions continued overleaf

Intrepreting the Results

- 1) Always make sure you read the results in the direction of the RED ARROW on the side of the glass tube.
- 2) The AlcoSense Single breathalyser has three lines on the tube, the further up the green passes the greater the alcohol level you are at.
- 3) When the crystals change green over each line you have passed the respective limit. e.g. if you pass the first line you are over 0.2‰BAC, the second line you are over 0.5‰BAC, the third line 0.8‰BAC etc.

Any amount of alcohol impairs your ability to drive. If you have any greening of the crystals or do not feel 100%, do not drive.


Common Drink Drive Limits

0.8‰BAC   English and Welsh Limit

0.5‰BAC   Scottish and Irish Limit

0.2‰BAC  Irish limit for professional and newly qualified drivers

A full list of limits is available at: www.alcosense.com

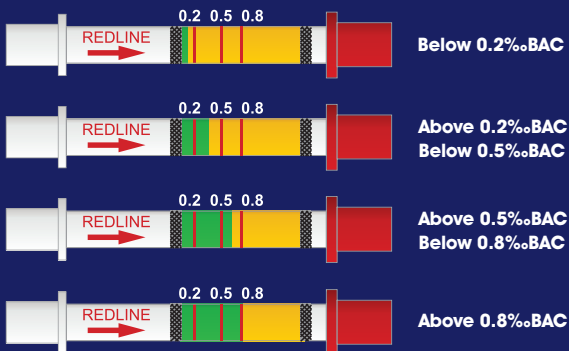

‰BAC = Per-Mil Blood Alcohol Content

Important:

- This breath tester provides a good indication of your alcohol level when used properly. The results are not conclusive and cannot be used to oppose results obtained from official law enforcement tests.
- Any amount of alcohol impairs your ability to drive. If you have any greening of the crystals or you do not feel 100% DO NOT DRIVE.
- This tester contains strong oxidizing agents. Handle with care, keep out of reach of children and dispose carefully after use in a waste bin. In case of accidental contact with eyes or skin rinse well immediately with water and seek medical advice.
- Store out of sunlight, between 5 and 40 degrees centigrade.
- If used after the expiry date the results may be unreliable.
- It is normal for the tube to become warm after use.
- The temperature of the tube must be between 5 and 35 degrees centigrade for the best results. Testing outside these temperatures may produce unreliable results.
- Results taken more than 5 minutes after use are unreliable because the crystals become gradually wetter from the atmosphere and affect the results.
- This tester can be used only ONCE.
- Your alcohol level can continue to increase for up to 90 minutes after consumption of your last alcoholic drink.